



HELEN

HELEN HOLDING GROUP

HELEN

HELEN HOLDING GROUP



Departure Destination | Saudi Arabia

FROM 2024



Destination Reaching | The World

TO 2025

HIFU LIPOSONIX

WWW.HELENHOLDING.COM - 920033794 - HELENHOLDING@OUTLOOK.SA

BUSINESS CLASS TICKET



9 2 0 0 3 3 7 9 4



Profile

On the stage, screen, the stars are always shining, always enjoy the beautiful youth like old legends .To know that they are not the goddess who don't eat the human things ,the same need to be maintain the skin so that could keep the beautiful in front of other people. So what is the skirmisher of anti-aging for those stars? The answer is Ulthera polar sonic lift (ultrasound).Different from traditional anti-aging equipment, it can act to reach the operation pull skin effect of Smash layer, no invasive treatment can achieve firming, wrinkles, shaping effect through time, take you back to the youth.



Specification parameter

Type	Focused ultrasound
	10MHz 7MHz 4MHz
pattern	Pulsed digital / Mode output
watt	180w
Therapeutic probe	DS10-1.5 DS7-3.0 DS4-4.5
	DS10-1.5 DS7-3.0 DS7-4.5 DS4-8.0 DS4-13
	5.0~25mm 1mm/step
Input power	AC 110V~220V~50Hz/60Hz

easy to
perform
very
effective





Principle

Using the most advanced ultrasonic energy, deep penetration to the deep dermis and SMAS layer, produce nearly ten thousand condensation point, make collagen, fibers produced and immediately shrink and stimulate collagen and new recombinant, constructing new collagen fiber network to enhance the elasticity from the bottom of the skin for the skin.



HIFU LIPOSONIX

11000





Principle

In addition, promote the popular sonic pull skin effect in the dermal layer of the skin, stimulate collagen hyperplasia and reorganization, compact profile and smooth lines, firming, wrinkle, shaping and other comprehensive effect, more focused energy deeper and stronger, more compact, more lasting effect, and Lund than the ICCT efficacy up for 1 to 3 years. The extreme nature, easy to pull, aged ten, is currently the latest generation of anti wrinkle skin care technology.



Principle

Ultrasonic to pull the skin will not burn the skin, the traditional pull technology temperature is not high, but also need to do skin cooling protection to prevent avoid the skin burns. And it's temperature as high as 68 to 72 degrees Celsius, produce the most suitable temperature for collagen, but it does not need protect the epidermis, and treatment process does not need anesthesia as it is no pain, the ultrasonic energy can across the skin, needn't to be worry completely burn the skin.




Treatment Effect


- 1、Tightening the skin of both cheeks
- 2、Improving the line of chin
- 3、Improving the skin, make it more smooth & delicate
- 4、Removing the Neck grain, prevent neck aging
- 5、Improving the elasticity of skin and make the contour more compact.
- 6、Tightening the forehead of coetaneous constituent
- 7、Eliminating wrinkles around the forehead, eyes and lips.




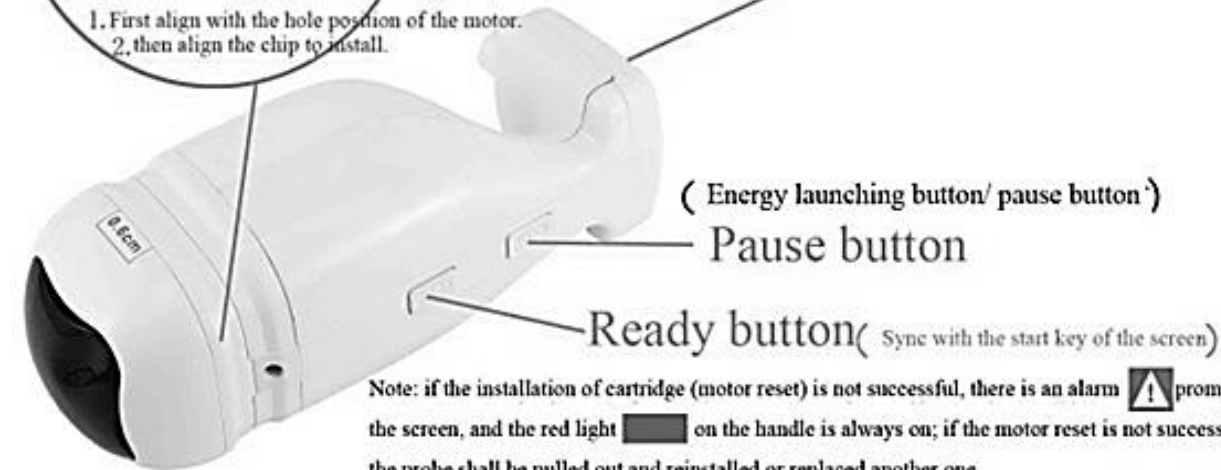
Parts Introduction

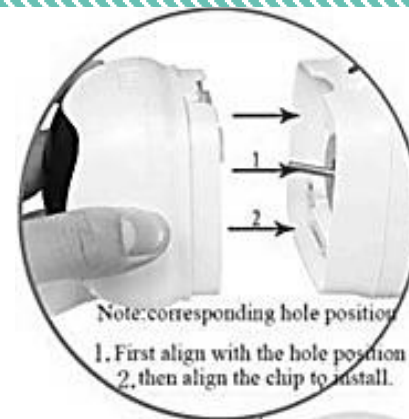

110100



(Blue light  is always on when launching energy)


(Green light  is always on when cartridge installed successfully)



Note: if the installation of cartridge (motor reset) is not successful, there is an alarm  prompt on the screen, and the red light  on the handle is always on; if the motor reset is not successful, the probe shall be pulled out and reinstalled or replaced another one.

Operation interface

Treatment mode represents the pattern of the energy emission output of the liposonic cartridge. The operating mode can only be selected when the instrument is in the suspended state, 576 output points in full style

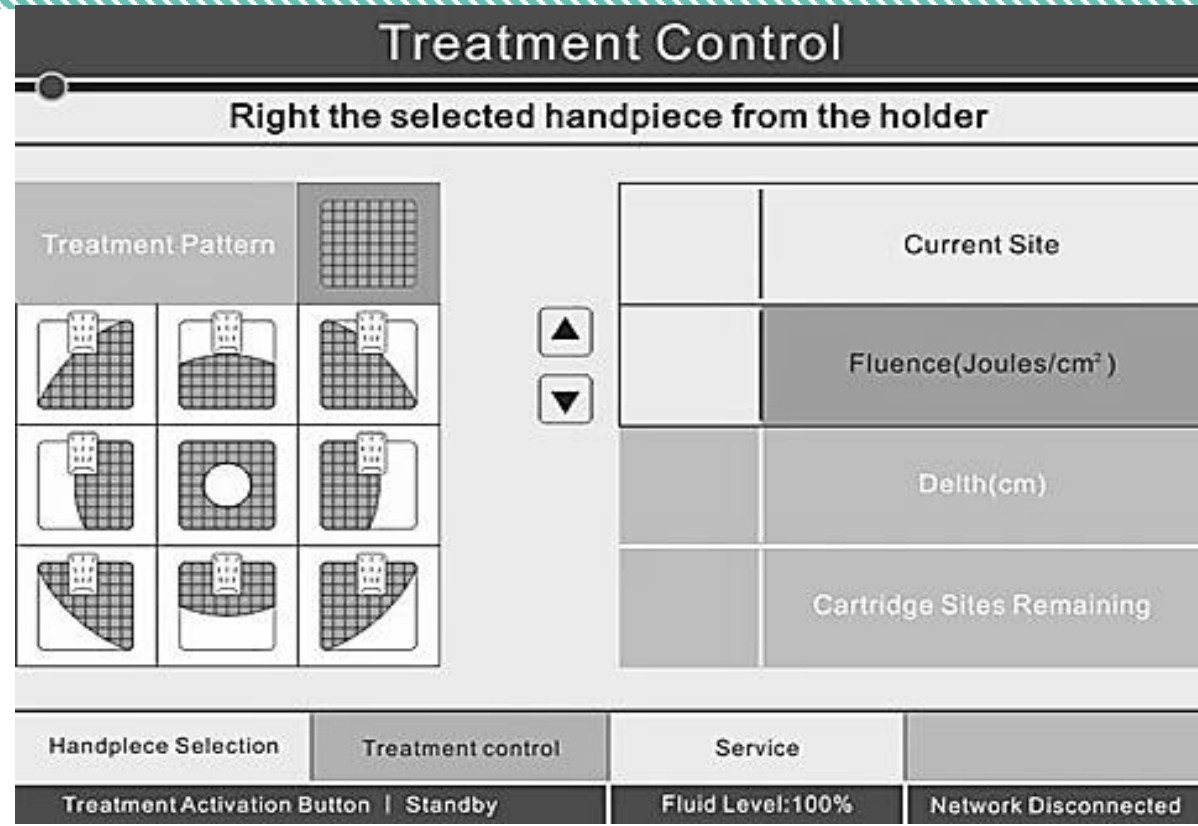

How often do the liposonic? How long will it take? Does it work?

Q:Obese customers do it according to the course of treatment. For severe local obesity, it takes 1-3 times to achieve the desired effect, and do it every other month. For the relatively small body, liposonic can be made only once, and it just need to take an hour, that will reduce the local size by 2-6 cm;

The clinical test results showed that up to 91.3% of the consumers improved their abdominal flatness significantly after treatment,and their waist circumference was 2 inches smaller on average. The body becomes slimmer and the waist curve returns to exquisite, which only needs 1 hour of treatment.

Does the customer need a break?

No breaks. Customers can resume normal activities immediately.



Common Failures

- 1、 There is no display on the instrument interface. Please check if the power plug is not connected or the power switch is turned on.
- 2、 The instrument is normal but there is no energy output or it is always on alarm. Please check if the handle cable is firmly connected to the host or the cartridge is installed in place.



Before Treatment

- Within 1 week before treatment, avoid taking aspirin or aspirin drugs, except for medical advice. Try to avoid alcohol, fish oil nutritional supplements and vitamin E.
- 1 week before treatment, please deactivate skin care products containing type vitamin A, or nursing products containing fruit acid. The products can be used 2 weeks after treatment.



After Treatment

- Keep the treatment area clean and dry,
- The surrounding skin can be cleaned with a wet towel.
- After string, check the residue of the disease, if there is residue, an early supplementation is necessary.



Parameters

Thickness of fat	above25mm	above23mm	above18mm	above14	above10mm
Therapeutic cartridge	1.6cm	1.3cm	1.0cm	0.8cm	0.6cm
Abdomen :1.6→1.3		Abdomen :1.0→0.8		Abdomen :0.8→0.6	
Legs :1.6→1.3		Legs :1.0→0.8		Legs :0.8	
Hip :1.6→1.3				Hand/back/chest:0.80.6	
All probe operation steps are from deep to shallow， such as（1.6→1.3）： First operation1.6cm， Then repeat the operation position of 1.6 probe with 1.3 probe.					





WWW.HELENHOLDING.COM

HELENHOLDING@OUTLOOK.SA

KAS- ABHA – ALMAHALA DIS - TEL | 920033794



HELEN

HELEN HOLDING GROUP