



HELEN

HELEN HOLDING GROUP

HELEN

HELEN HOLDING GROUP



Departure Destination | Saudi Arabia

FROM 2024



Destination Reaching | The World

TO 2025

HAYDRAFACIAL

WWW.HELENHOLDING.COM - 920033794 - HELENHOLDING@OUTLOOK.SA

TICKET
BUSINESS CLASS



9 2 0 0 3 3 7 9 4



PLATINUM SUPER HYDRAFACIAL

First of all, let us express our gratitude to the customers who have purchased our instruments. This manual is written for customers to use this instrument better. I hope that the company's products and services will bring you respect and courtesy!

Product Description

Black Pearl - The most comprehensive introduction in history

1. Bubble head: vacuum negative pressure, with negative pressure to take away the dirt from the face, clean the face.
2. Shovel skin knife: clean, exfoliate, moisturize the skin.
3. Spray gun: high-pressure oxygen injection to replenish water, deep muscles into the skin layer, easier to absorb through the fine pores.
4. Wave head (face + eyes): Import the product to make the product more absorbed, fine pores, and smooth skin.
5. Eye head: improve the fine lines of the eyes, tighten and lift.
6. Hot and cold head: shrink pores, block nutrition, calm and repair.
7. Girl's line: Gua Sha, detoxification, lifting and firming contours, introduction of essence, shaping and diminishing fine lines
8. Radio frequency head: enhance firmness and promote the re-growth of collagen.


easy to
perform
very
effective



Screen


New technology slidabe LCD screen ,Touch “ENter”

100000

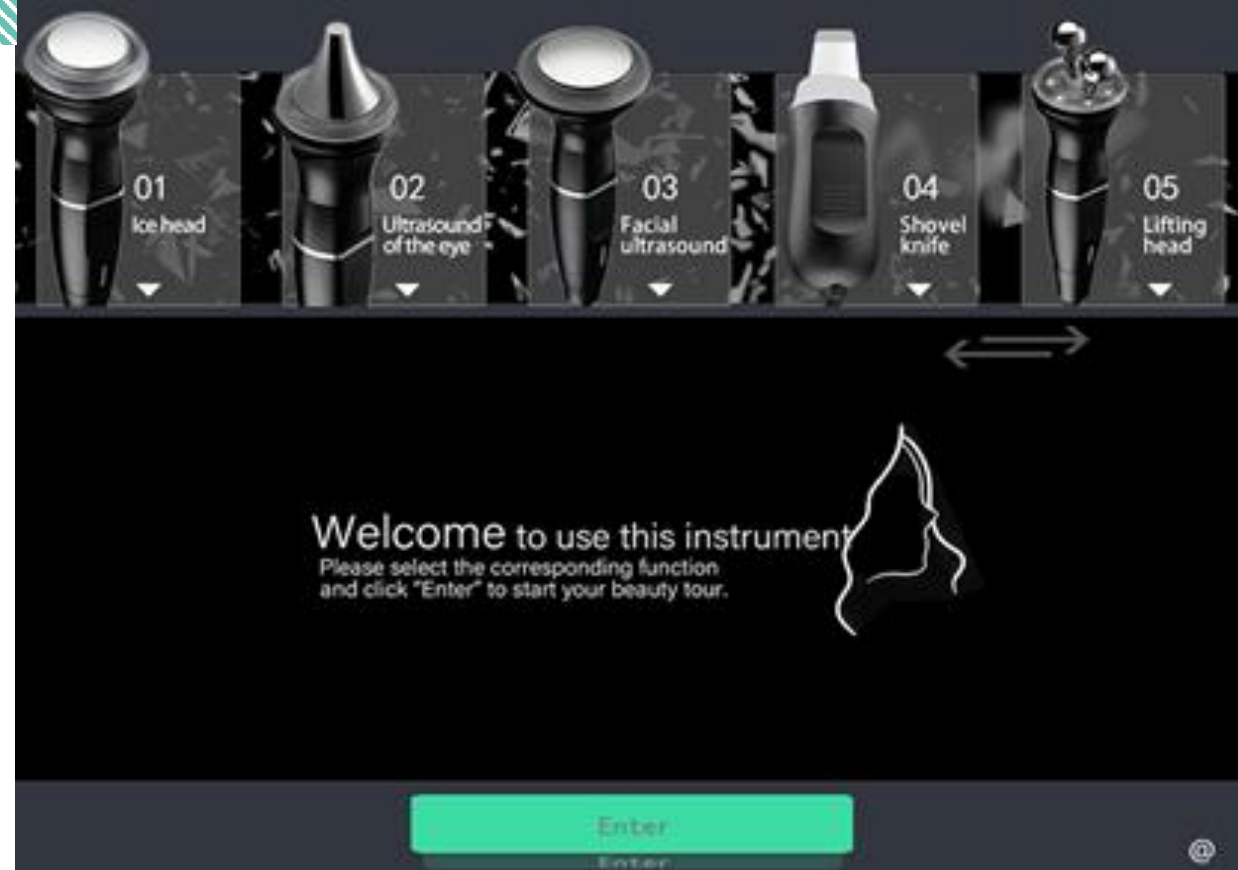


Cool and warm head

The ice treatment allergy ,enhance immunity ,Skin tightening ,shrinks pores ,tightens skin , removes wrinkles & Promotes collagen hyperplasia , eliminates redness and sensitivity. Accelerate product absorption.

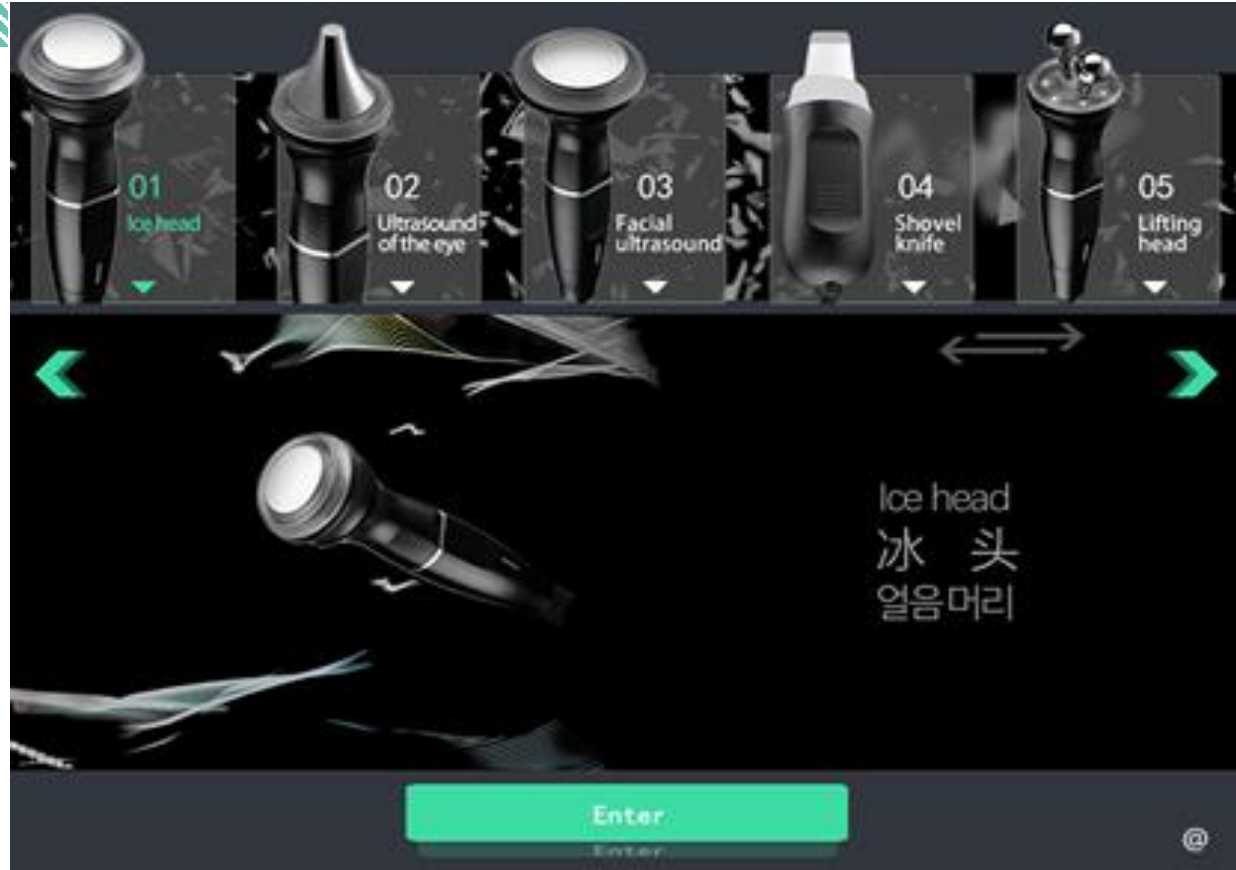


RF Lifting head

Very effective in tightening the skin and stimulating blood circulation in the face and neck

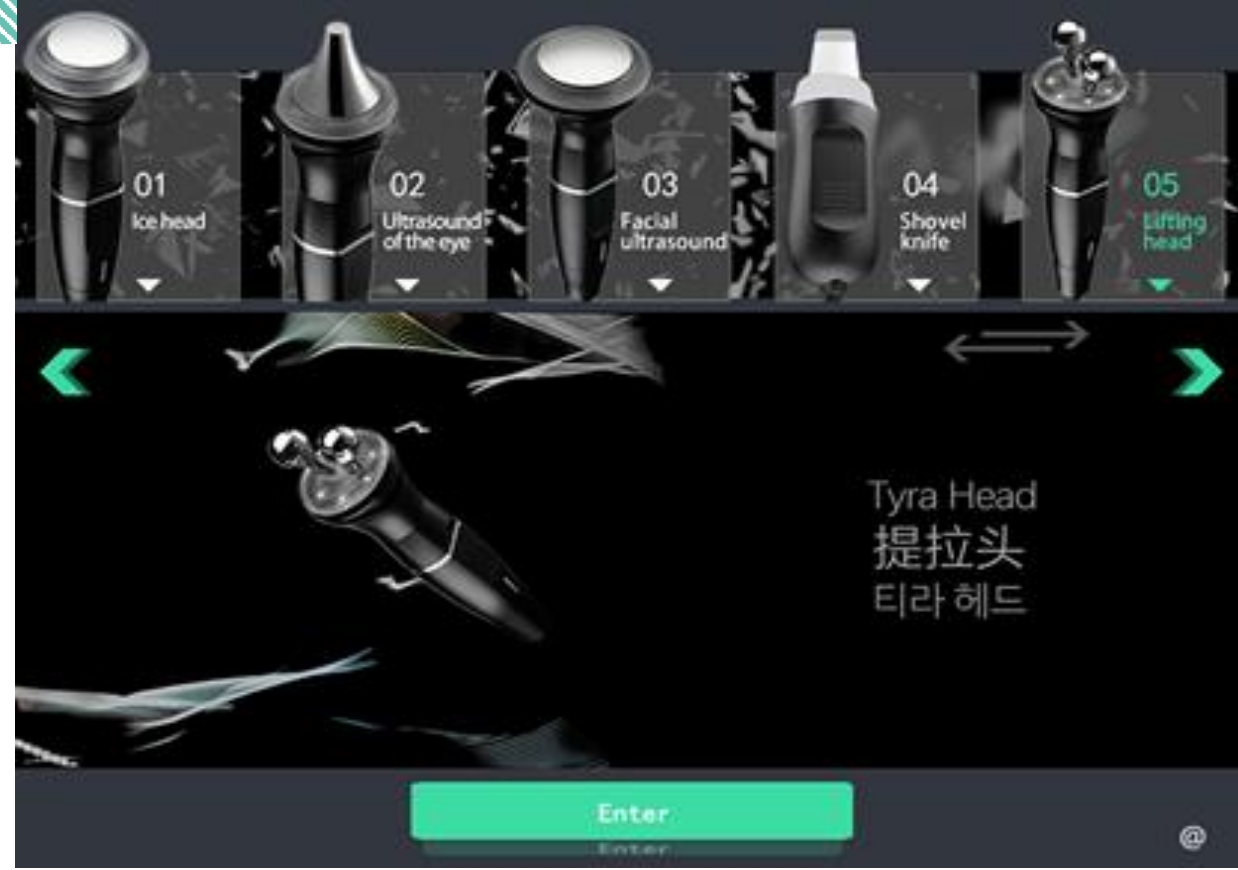

Eyes ultrasound head

Helps reduce dark circles around the eyes –
Supplying the face, cheeks and lips

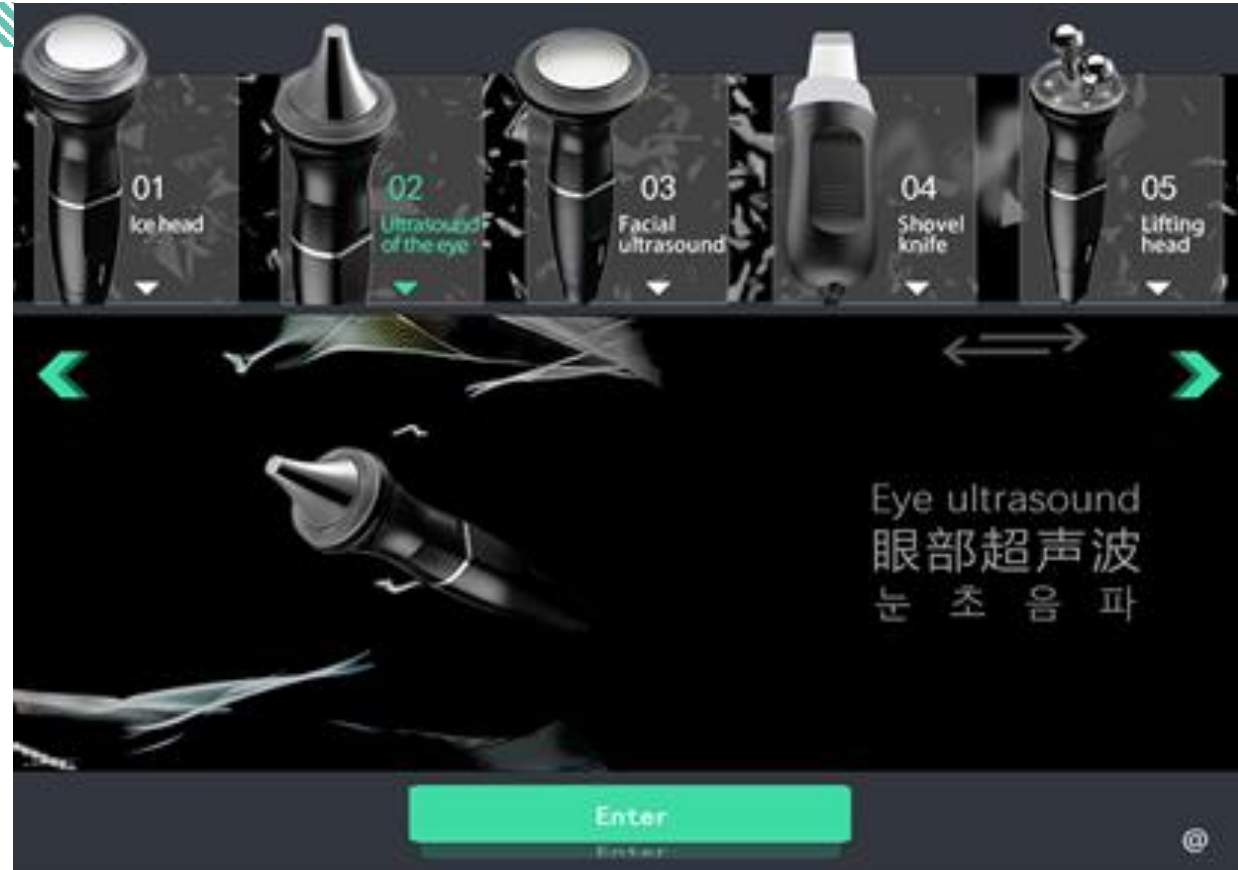

E Facial ultrasound head

Promotes collagen production in the face and neck area



Miro Bubble cleaner head

Immediate result - removes blackheads and dead cells,
imparts clarity and purity



CRESTAL


cleaner head

Microdermabrasion, which needs no introduction, removes a lot of fat with effective suction power




Beauty lifting Head

An effective force to tighten the chin area
and define the jaws without hassle or pain

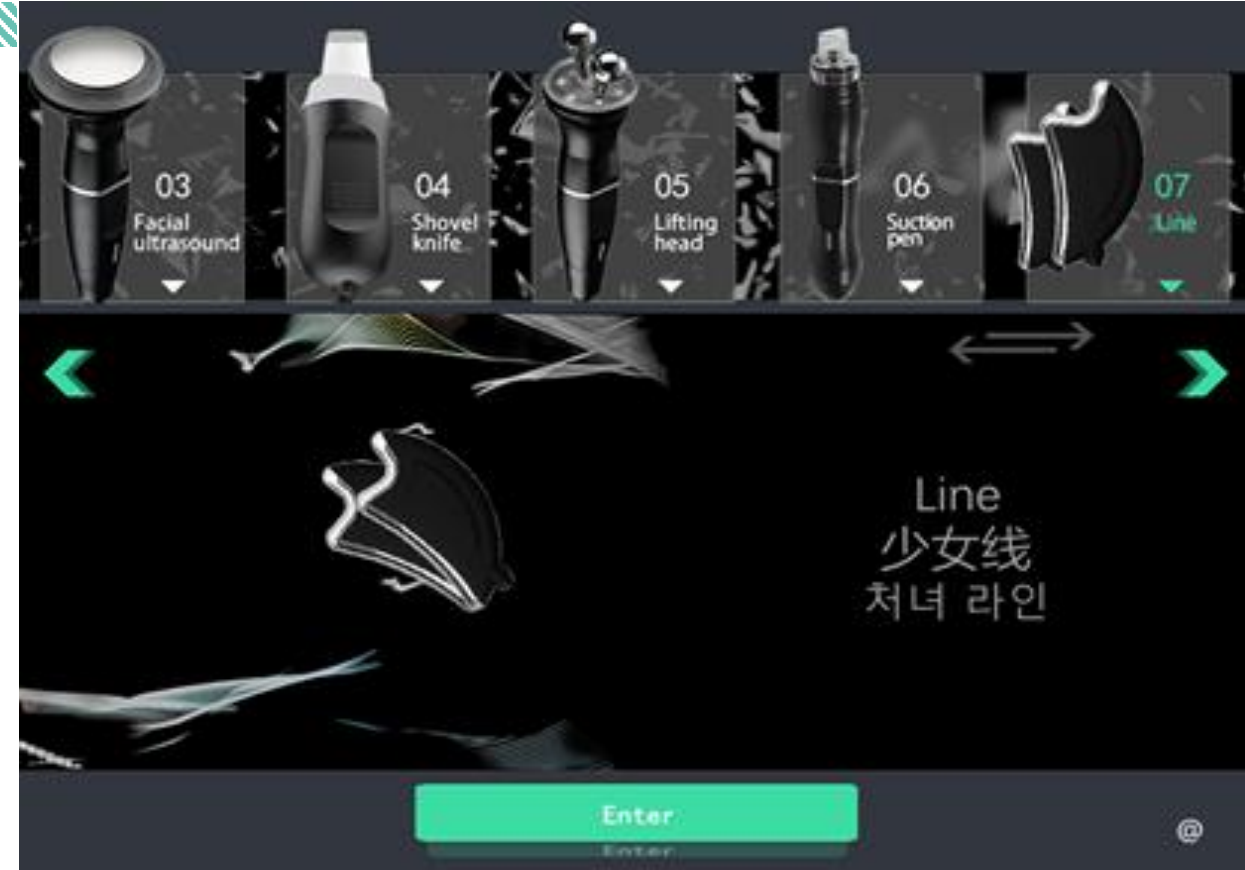

Water oxygen head


A soft touch to conclude a perfect hydrafacial session, as the device oxygenates the water directly

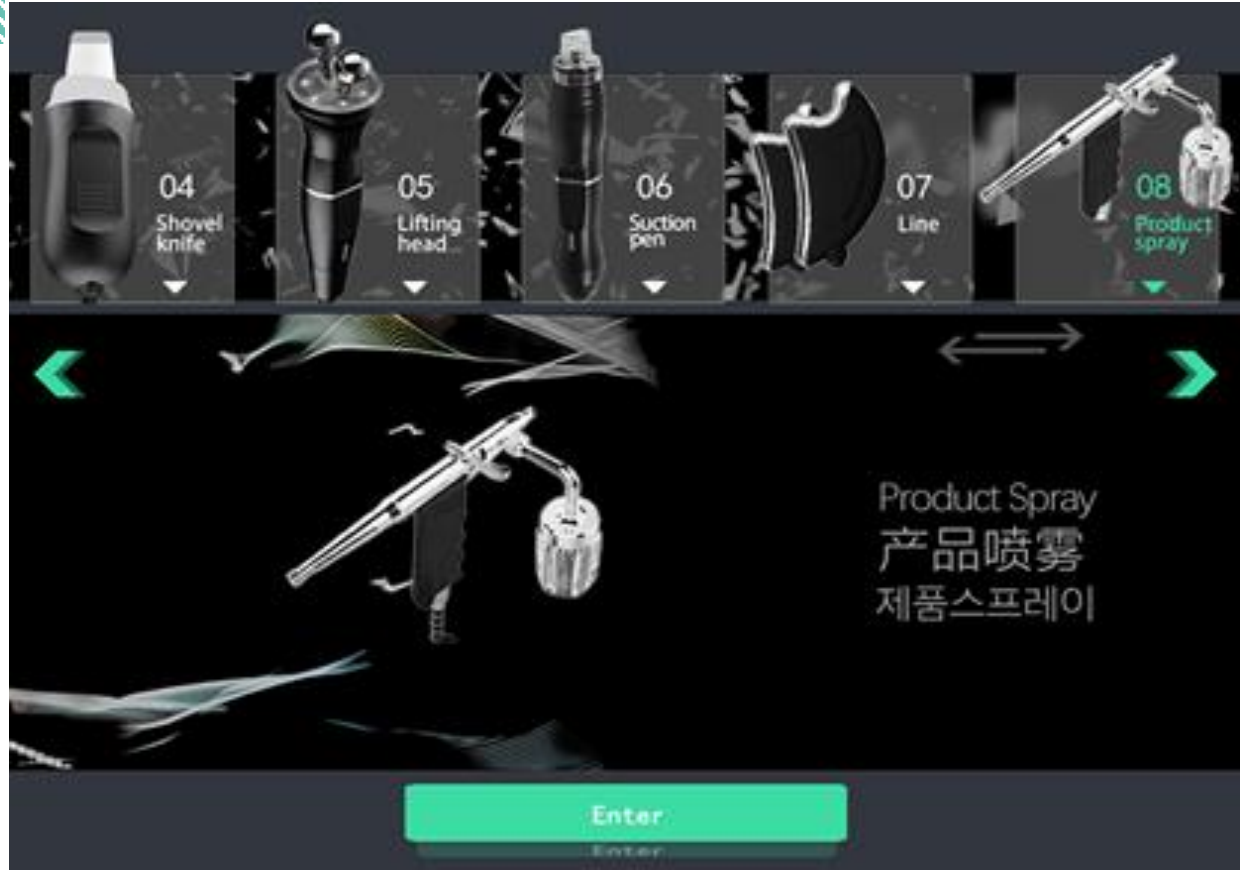

Operation Steps



START

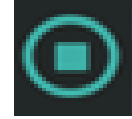

STOP



This is to choose the strength of energy



This is the time to choose the controller to work



Precaution

- 1: Before use, please fill the solution bottle with proper purified water or nutrient solution, and do not use tap water.
- 2: Pay attention to the liquid level of the waste bottle, please discard it in time to prevent damage to the instrument.
- 3: Use with caution if the skin stratum corneum is thin or the cheekbones are severely red.
- 4: People who have undergone facial surgery or deep exfoliation, but have not yet reached the recovery period, use it with caution.
- 5: After the operation, it is necessary to strengthen the hydration, and apply the hydrating mask continuously for a week. Do not use functional hydrating products.



Unsuitable people

1. People with pacemakers or other similar electronic devices implanted in their bodies. ne / Oratane).
2. There are fillers inside. If gold wires are implanted on the face, it is not recommended.
3. Patients with severe heart disease.
4. The surgical wound has not healed.
5. Patients with advanced tumors.





WWW.HELENHOLDING.COM

HELENHOLDING@OUTLOOK.SA

KAS- ABHA – ALMAHALA DIS - TEL | 920033794



HELEN

HELEN HOLDING GROUP