



HELEN

HELEN HOLDING GROUP

HELEN

HELEN HOLDING GROUP



Departure Destination | Saudi Arabia

FROM 2024



Destination Reaching | The World

TO 2025

ELEIT LASER +

WWW.HELENHOLDING.COM - 920033794 - HELENHOLDING@OUTLOOK.SA

BUSINESS CLASS TICKET





9 2 0 0 3 3 7 9 4



Theory of Elite Titanium laser Plus +

3 wavelength laser com in the 808nm laser, 755nm laser and 1064nm laser in one machine. Which ensure the machine can do the hair removal for any skin type and any hair type and all hair colors. And the treatment result is very good.


for

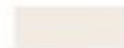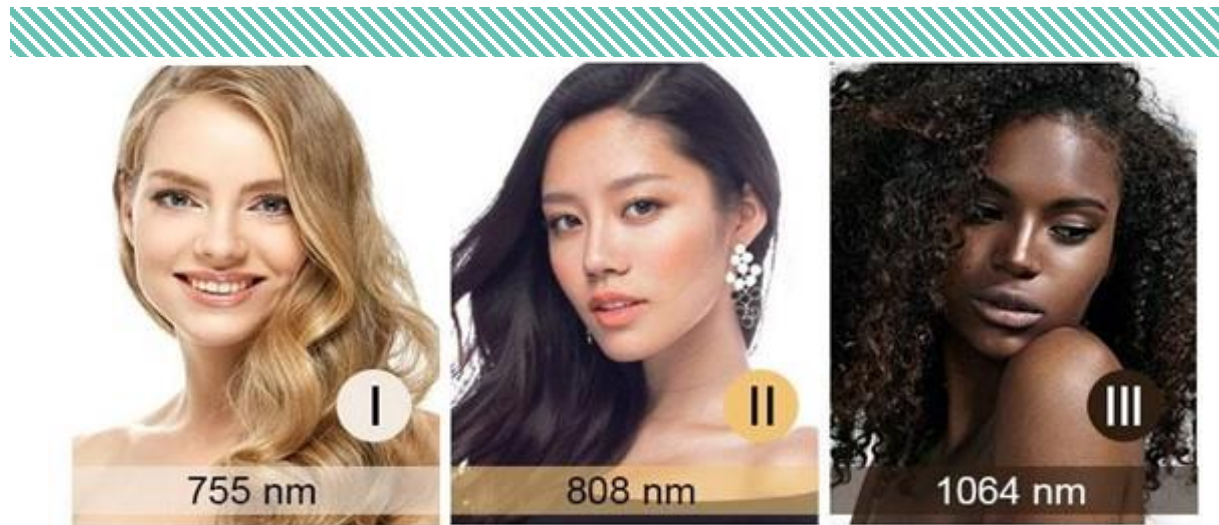

755 nm for white skin (fine, golden hair removal);



808 nm for yellow neutral skin (brown hair removal);

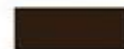

1064 nm for black skin (black hair removal)

easy to
perform
very
effective



WOLFOO



WORKING THEORY

It is best effect of treatment when the hair is growing. In the period of growth stage, The pigment in hair follicle will be absorbed completely. In that time, the follicle is weakest, the effect of hair removal is best. You will see the effect when you have 2 times treatment. Not only can remove the extra hair, but also it is applied for changing the shape of beard.

Helen

MEDICAL HOLDING COMPAN



Helen

Helen

MEDICAL HOLDING COMPANY

Hair removal principle

Make Your Beauty Business More Easier



The laser selectively acts on the melanin in the hair follicle, destroying the hair-growing area

Natural hair loss, prevent regeneration, and achieve the purpose of hair removal

Hair removal also stimulates collagen regeneration, shrinks pores and makes skin firm and smooth



The process of hair removal first time

1. consulting - Can visit to our company, the beautician will make plan for you according to your specific situation.
2. Confirming the skin - The beautician will make a confirmation for your skin, the density of hair.
3. Painting cooling gel - Clear the hair, and paint the cooling gel.
4. shining - Customer should wear goggle, whole treatment only use 3 minutes.
5. cooling - Clear away the cooling gel, using cooling bag.
6. Calming down - Paint the beauty fluid, and protect the skin.



Helen

MEDICAL HOLDING COMPAN

OVER VIEW

10000



Security overview

According to the classification of electrical safety, this instrument and equipment belongs to I category, and can be used for mobile general equipment. The following is an overview of the security precautions:

- The non-company's professional and technical personnel shall not open the enclosure of the equipment under any circumstances.
- Do not move the device at will while working and lock the casters.

Do not repair the equipment yourself if it does not work properly. should immediately contact our after-sales customer service.

- There is no internal storage of your required repair accessories.



Function Of Instrument

The 808 freezing point hair removal technique, which originated in Germany, is based on the principle of selective photothermal kinetics, through the rational regulation of wavelength, energy and pulse width, so that light energy passes through the skin surface to the root hair follicle of the hair, and is absorbed and converted into heat energy that destroys the hair follicle tissue, thus causing the hair follicle tissue to grow slowly or not. The extreme freezing point depilation system uses unique freezing techniques to cool the local epidermis to 5 degrees Celsius, as in summer

Snowflakes with cool ice come to cool off; overcome the early hair cold touch color side effects, as well as skin damage, heavy equipment, complex operation and other shortcomings. At the same time no damage, painless sense, is the current safer, faster, more lasting hair removal technology.




Heleen

MEDICAL HOLDING COMPAN

OVERVIEW

10100

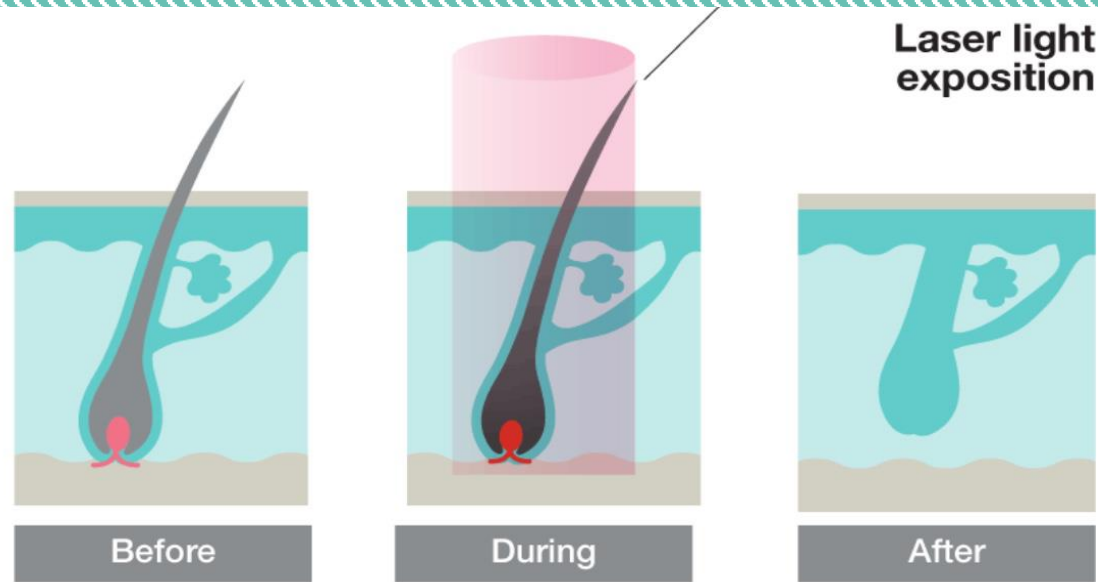

Cooling Technology






Cooling technology provides cooling before, during and after treatment to protect the epidermis and reduce the temperature of the initial epidermis. This can reduce the user's discomfort.

cooling most importantly, the energy of depilation can be added to the maximum without increasing the possibility of side effects.

Dense hair (30-40 hairs per CM^2) produces higher temperatures because of more melanin per CM^2 ,. Because the surrounding tissue produces excessive heat, it is easy to cause tissue damage. therefore, the higher the hair density, the lower the energy



SKIN COLOR

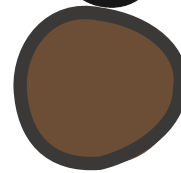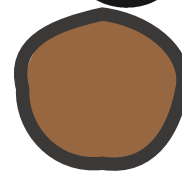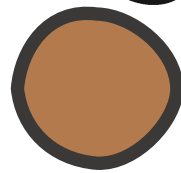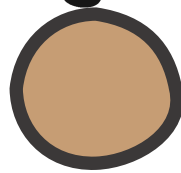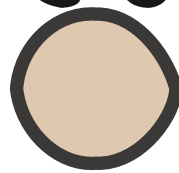



WWW.HELENHOLDING.COM

HELENHOLDING@OUTLOOK.SA

KAS- ABHA – ALMAHALA DIS - TEL | 920033794



HELEN

HELEN HOLDING GROUP