



HELEN

HELEN HOLDING GROUP

HELEN

HELEN HOLDING GROUP



Departure Destination | Saudi Arabia

FROM 2024



Destination Reaching | The World

TO 2025

FRACTIONAL R.F

WWW.HELENHOLDING.COM - 920033794 - HELENHOLDING@OUTLOOK.SA

BUSINESS CLASS TICKET



9 2 0 0 3 3 7 9 4



What is the Fractional Free - electron laser gold RF microcrystal?

The Fractional Free-electron laser gold radio frequency microcrystal is the ingenious combination of free electron laser micro crystal and radio frequency. The two word of "gold" originates from the microcrystalline gold coating and the coating is also golden yellow. At the time of treatment, the doctor according to the crystal position on Problems and treatment, adjust the depth of penetration and micro crystalline radiofrequency energy, then in the electronic control system, at the same time, dozens of insulating ceramics penetrate the skin quickly, from the micro crystal tip radiofrequency energy, then quickly exit, so the cycle until the treatment is finished, finally apply cosmetic ingredients.

Action Mechanism

- 1, micro crystal mechanical stimulation, coupled with the biological effects of radio frequency and thermal stimulation, together to stimulate the skin's self repair system, to promote metabolism, improve microcirculation, the initiation of collagen and other new and rearrangement;
- 2, microcrystalline penetration, open the skin quickly absorption channel, easy to cosmetic ingredients into the skin;
- 3, the radio frequency energy emitted by the micro crystal tip can selectively destroy the hair follicles, sebaceous glands and sweat glands, inhibit inflammatory reaction and activate the anti-inflammatory and repair system.

easy to
perform
very
effective

Woloo



WORKING THEORY OF MICRONEEDLE RF

Fractional Free-electron laser gold radio frequency RF technology allows effectively treatment of deeper dermis, epidermis, and upperdermis. Free-electron laser and RF energy is emitted at the end of fractional RF microneedle; this will treat target area-deeper dermis without damage of epidermis. The fractional RF is useful for minimal invasive epidermal rejuvenation. RF energy does not target pigment-making cells in the skin. Minas cures and improves wrinkle, scar, skin rejuvenation, and skin tightening safely and efficiently with minimized PIH, using RF energy.

1. Microneedle cartidge makes perfect contact to the skin;
2. Miconeedle smoothly penetrate the skin with less pain;
3. Collagen regeneration & new elastic production process begins.

Helen

MEDICAL HOLDING COMPAN



FUNCTION OF MICRONEEDLE RF MACHINE

1. Smooth away wrinkles and firming skin: crow's feet, head lines, neck lines, eye lines
2. Skin rejuvenation bright: large pore, skin dull, rough, uneven skin tone
3. Flattened scars: traumatic marks, stretch marks, obesity pattern
4. Pox acne and comedos: acne bar marks, depression pox pits, acn




Advantage of Microneedling RF treatment

1. Anti-wrinkle, firming the skin, improve pseudo wrinkles, dissolving fat, shaping and lifting.
2. Rapidly improve the dull and lusterless symptom, enhance dry skin and sallow complexions, brighten the skin and make it more tender.
3. Actively promote facial lymph circulation, solve the problem of skin edema.
4. Lifting and tightening the skin, effectively solving the problem of facial sagging, shaping delicate face, repairing stretch marks.
5. Removing black rim of the eye, eye bags and wrinkles around the eye.
6. Shrinking the pores, repairing blain scar, calming skin.



Helen

MEDICAL HOLDING COMPAN

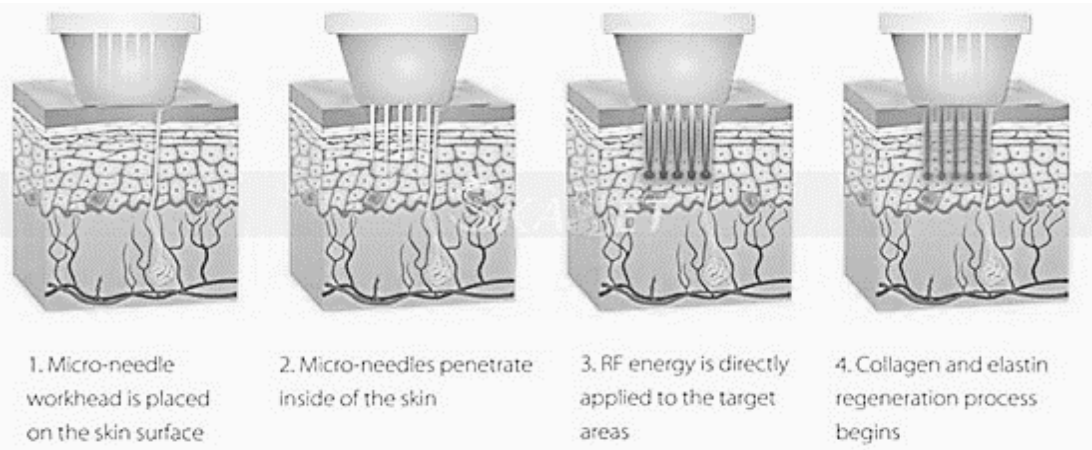

Valon

Parts Introduction


10000

F-face

F

B-body

B

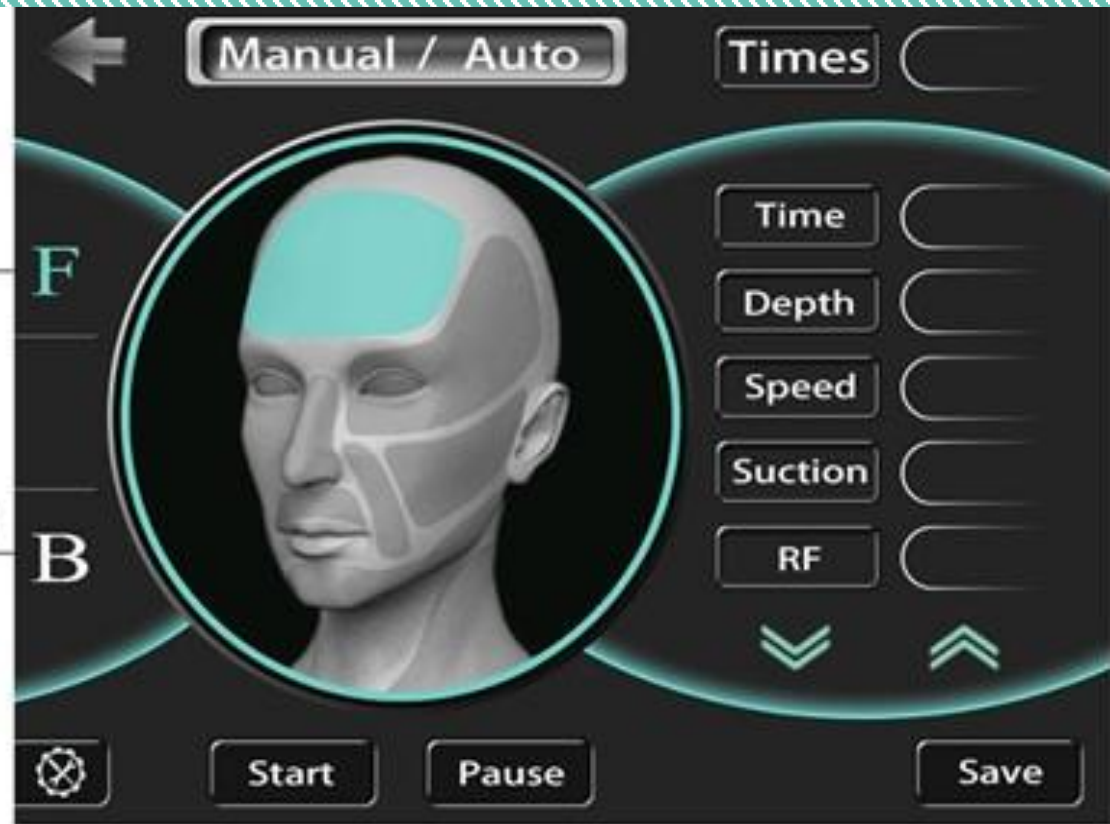

Parts Introduction


RF
Microneedle
handle



110000

Parts Introduction

110100

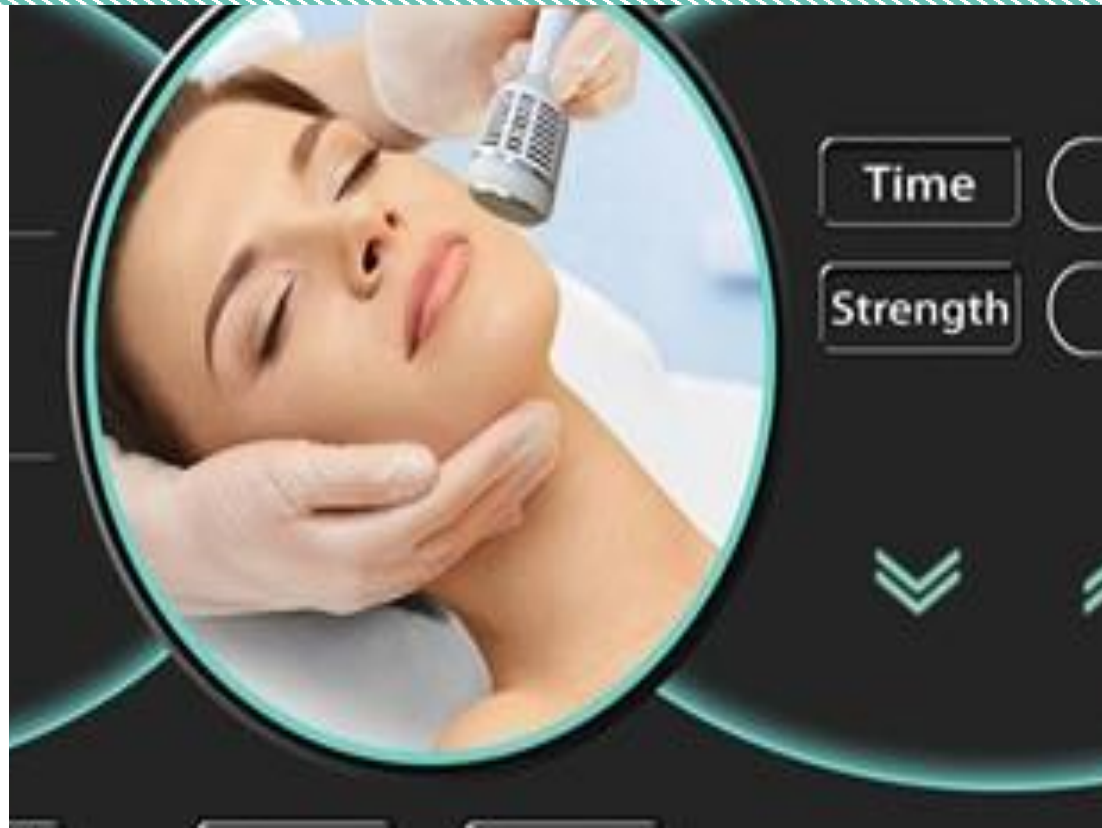

Technical specifications:

output frequency	2-4M
output power	10-200W
Therapeutic probe	10p 25p 64p
input voltage	AC 110V~220V~50Hz/60Hz



Please note

The RF energy value ranges from 5-150. It is recommended to start with the minimum energy of 5 for facials. The larger the energy, the better the effect, but the energy size needs to be selected according to the energy the customer bears.



Before Treatment

- Avoid exposure 2 weeks before treatment, apply sunscreen prevention.
- Within 1 week before treatment, avoid taking aspirin or aspirin drugs, except for medical advice. Try to avoid alcohol, fish oil nutritional supplements and vitamin E.
- 1 week before treatment, please deactivate skin care products containing type vitamin A, or nursing products containing fruit acid. The products can be used 2 weeks after treatment.



After Treatment

- After treatment, the customer cannot immediately touch the local to prevent bacterial infections; the treatment area uses antibiotic or non-antibiotic ointment according to the doctor's order to prevent bacterial infection.
- Keep the treatment area clean and dry, avoid water within 3-7 days. The surrounding skin can be cleaned with a wet towel.
- After the crusting, it is not necessary to force zushing to prevent damage to the skin, extend the cure time.
- After string, avoid sunlight or using sunscreen to prevent pampine. If a pigmentation occurs, apply the spot cream, or the pigment naturally disappear after 3 months or more.
- After string, check the residue of the disease, if there is residue, an early supplementation is necessary.



Parameters

Operating part	depth	Speed, i.e., energy output time	suction	RF size
forehead	0.5-1.0mm	0.2-0.3	1-2	5-50W
Face	1.0-1.5mm	0.2-0.3	1-2	5-50W
Eye	0.5-1.0mm	0.2-0.3	1-2	5-50W
chin	0.5-1.2mm	0.2-0.3	1-2	5-50W
nose	0.5-1.0mm	0.2-0.3	1-2	60-100W
neck	1.0-1.5mm	0.2-0.3	1-2	60-100W
Pox pits	1.0-2.0mm	0.2-0.3	1-2	60-100W
scar	1.0-2.0mm	0.2-0.3	1-2	60-120W
belly	1.0-3.0mm	0.2-0.3	0-1	60-120W





WWW.HELENHOLDING.COM

HELENHOLDING@OUTLOOK.SA

KAS- ABHA – ALMAHALA DIS - TEL | 920033794



HELEN

HELEN HOLDING GROUP